		B2020-40STFX	B2020-40STMX	B2040TI
1	Flat point screw M8	V03I-0816	V03Z-0816	V03I-0812

Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screw	Torques (Nm)	Screw	Torques (Nm)
V03I-0816 and V03I-0812	13	V03Z-0816	15

- Assemble the screws using normal (Ref. : Z31-243) locking compound
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- Use a damp cloth to clean the product

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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B2040TI

Designation

Tubular adapter with light alloy tube \varnothing 30 mm

- | | | |
|-------------------------------|----------------|-----------------|
| ⌚ B2020STFX : Light alloy | Weight : 193 g | Length : 200 mm |
| ⌚ B2040STFX : Light alloy | Weight : 333 g | Length : 400 mm |
| ⌚ B2020STMX : Stainless steel | Weight : 236 g | Length : 200 mm |
| ⌚ B2040STMX : Stainless steel | Weight : 358 g | Length : 400 mm |
| ⌚ B2040TI : Titanium | Weight : 308 g | Length : 400 mm |

Purpose

Structural element used to make endoskeletal prostheses of the lower limbs

- ⌚ B2020STFX et B2040STFX : Weight limit : 100 kg
- ⌚ B2020STMX et B2040STMX : Tested in accordance with ISO 10328 P6 136 kg
- ⌚ B2040TI : Weight limit : 136 kg

Adjustments

Tilt amplitude : +/- 9° compared to the reference axis.

Set the desired tilt using the 4 screws, in accordance with the limits set.

Lock the assembly using (Ref. : Z31-243) locking compound on the screws.

The tube can be cut again in length according to your needs.

Do not grip the tube in a vise !

Use tube cutter (Ref. : ZOC103) by applying it in comparison with the tube at right angles. Deburred carefully inside and outside of the tube with the reamer (Ref. : ZOC104).

Insert the tube to the stop of the adapter.