		BA350STFX	BA350STMX
1	Flat point screw M8 L16	V03I-0816	V03Z-0816

Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screw	Torques (Nm)
V03I-0816	13

Screw	Torques (Nm)
V03Z-0816	15

- Assemble the screws using normal (Ref. : Z31-243) locking compound
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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BA350STFX/STMX/Ti

Designation

Tubular with adapter Ø 30 mm tube for pyramid

- | | | |
|-----------------------------|----------------|-----------------|
| BA350STFX : Light alloy | Weight : 83 g | Length : 100 mm |
| BA350STMX : Stainless steel | Weight : 207 g | Length : 100 mm |
| BA350Ti : Titanium | Weight : 313 g | Length : 100 mm |

Purpose

Structural element used to make endoskeletal prostheses of the lower limbs

- BA350STFX : Weight limit : 100 kg
- BA350STMX : Weight limit : 136 kg
- BA350Ti : Weight limit : 136 kg

Adjustments

Tilt amplitude : +/-9° compared to the reference axis.

Set the desired tilt using the 4 screws, in accordance with the limits set.

Lock the assembly using (Ref. : Z31-243) locking compound on the screws.

The tube can be cut again in length according to your needs. The adapter can be shortened by up to 35 mm maxi.

Use tube cutter (Ref. : ZOC103) by applying it in comparison with the tube at right angles.

Deburred carefully inside and outside of the tube with the reamer (Ref. : ZOC104).