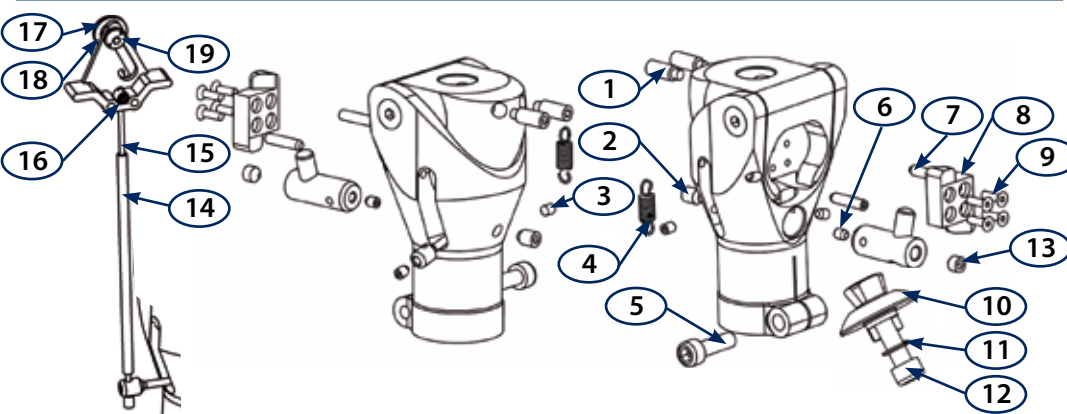
ST AIRFLEX

1	Rotation adjustment screw	8	Stop	15	Steel wire
2	Locking adjustment screw	9	Locking stop screw	16	Clamp
3	Point screw	10	Connector	17	Knurled nut
4	Return spring	11	SPS washer	18	Axis
5	Tube locking screw	12	Locking connector screw	19	Nut
6	Lever locking screw	13	Locking screw		
7	Spring screw	14	Nylon tube		

Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screws	Torques (Nm)
V02I-0620 (5)	10
V03TI-0616 (1)	5
V02I-0825 (12)	15

- ↪ Mount the tube on the stop of the bore bottom
- ↪ Lubricate lock system with molyckote Z10
- ↪ The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ↪ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ↪ All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- ↪ Stock the product in a dry place, in its original packing (on which the batch number is marked)
- ↪ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- ↪ Use a damp cloth to clean the product

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in incomplete forms cannot be dealt with.

EC markings

JANTON hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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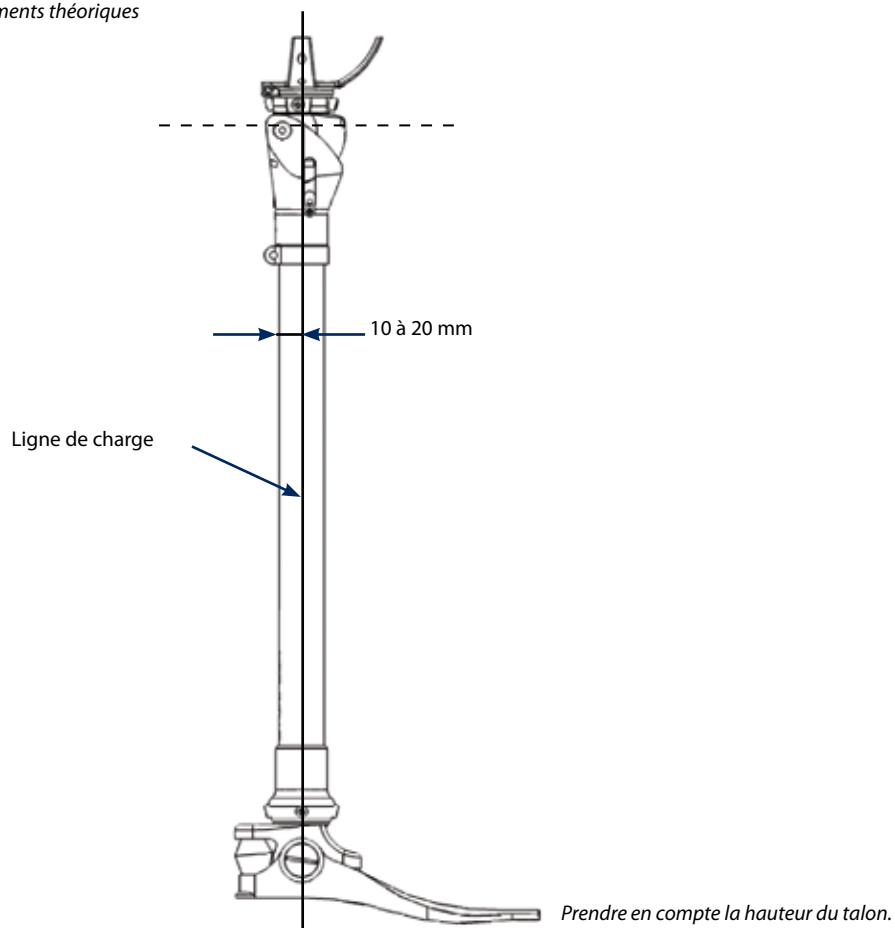
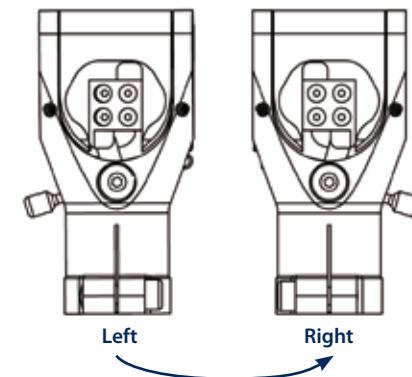
Adjustments

Adjustment of reverse unlocking side

(See spare parts)

Use LOCTITE 243 for definitive assembly (Ref. : Z31-243)

- Unlock the knee and unscrew screw (3)
- Take down spring (4)
- Take out screw (2) then (6)
- Change lever and spring side unscrewing screw (7)
- Reassemble the spring on the lever and screw (6)
- Return and reassemble stop (8) after unscrewing screws (9)
- Lock the knee and adjust it with screws (2) and (3)

Adjustment of locking play

(See spare parts)

Use LOCTITE 243 for definitive assembly (Ref. : Z31-243)

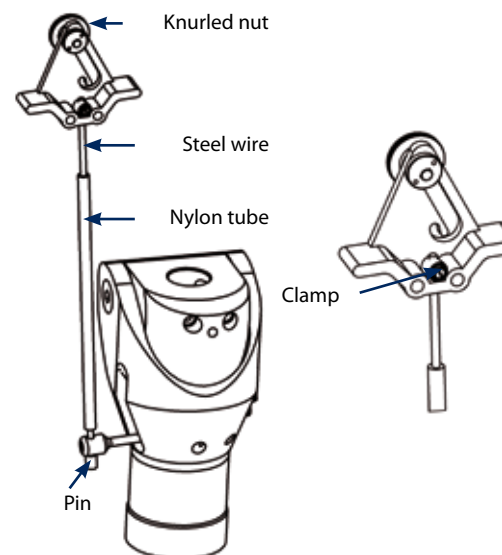
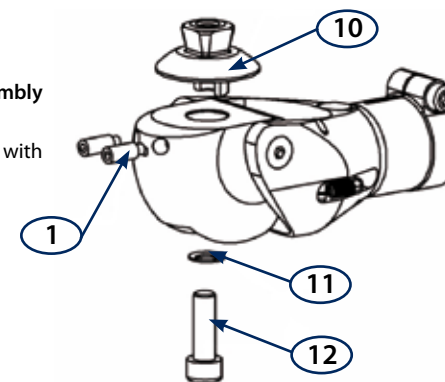
- Lock the knee
- Unscrew screw (3)
- Screw screw (2) up to be in contact with stop (8)
- Control the locking, adjust if necessary
- Lock the adjustment with screw (3)

Adjustment of proximal attachment

Rotation maximum : $\pm 5^\circ$

Use LOCTITE 243 (Ref. : Z31-243) for definitive assembly

- Unscrew screw (12)
- Adjust the rotation of the connector (10) with screws (1)
- Screw screw (12)

Fitting of the unlocking lever

- Place and laminate the nylon tube (14) and the knurled nut (17)
- Settle the handle on the socket
- Put the steel wire (15) into the lever and next in the handle. The pin of the steel wire must be in contact with the lever
- Adjust the stretch of the wire, screw the clamp (16) and glue it with LOCTITE 243 (Réf. : Z31-243) Cut the surplus of wire

Compatibility

- Proximal attachment : connector of the STARFLEX range
- Distal attachment : Ø 30 mm tube
- For the unlocking lever : ref. C102001P and C1110

Connectors in the STARFLEX range (upper section)

CA1
Male pyramid


CA3
Connector
for temporary fitting

CA7
Rotary adaptor with anchor
for cementing, large branches

Setting up

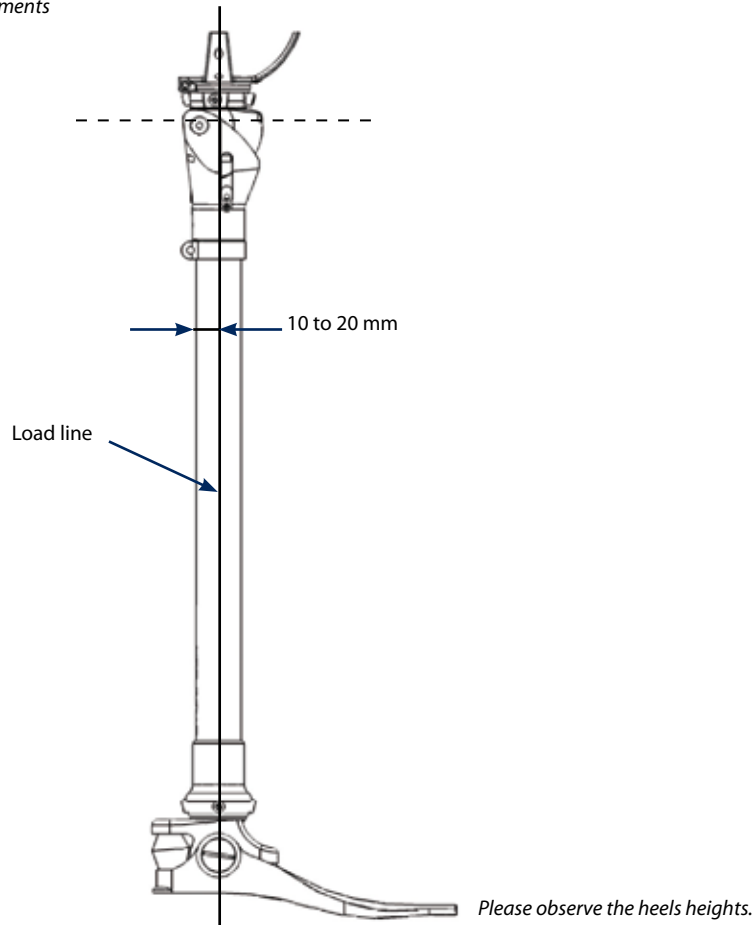Theoretical alignments

Processing

In order to optimise your assemblies, we recommend that you use this knee in conjunction with those of the JANTON range in identical class.

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- ↪ Patient with moderate activity. Limited walking area. Indoor/outdoor use