Screw	Hex key	Torque
1	5 mm	7 N.m
2	8 mm	10 N.m
3	5 mm	
4	4 mm	
5	4 mm	
6	2 mm	
7	2 mm	

**Restrictions one use / Precautions for use**

- ↪ The knee C29 must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ↪ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ↪ All assembly and adjustment work must be carried out by qualified personnel

**Cleaning / Storage**

- ↪ Store the product in a dry place, in its original packing (on which the batch number is marked)
- ↪ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- ↪ Use a damp cloth to clean the product

**Guarantee**

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.  
*N.B.: In the event of a complaint, please enclose a return form correctly filled in; incomplete forms cannot be dealt with.*

**EC markings**

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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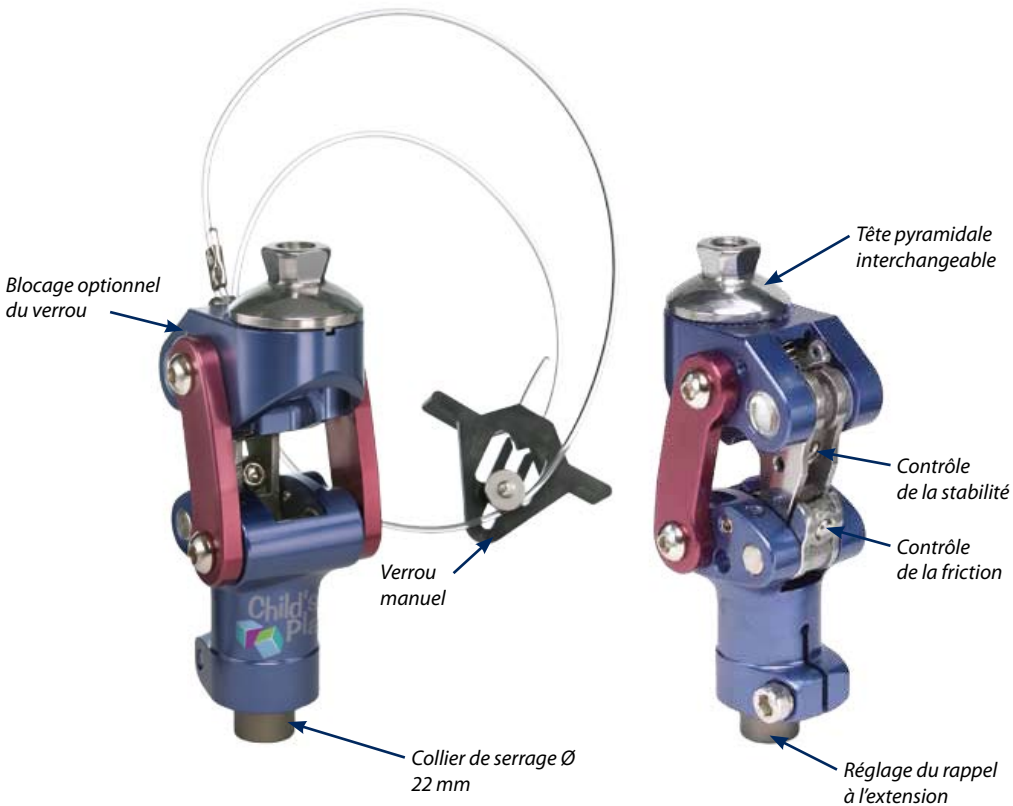
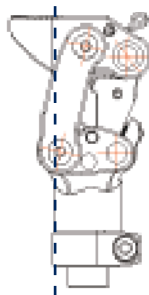
## Adjustments

### Extension spring assist (Screw 3)

- ⌚ Loosen the screw (1). Disassemble the knee tube
- ⌚ Unscrew the screw (3) to reduce the recall to the extension
- ⌚ Tighten the screws (3) to increase the recall to the extension

#### Warning !

Do not tighten the screw (3) completely

### Friction control (Screw 4)

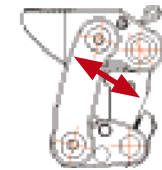
- ⌚ Unscrew the screw (4) to reduce friction swing phase
- ⌚ Tighten the screws (4) to increase the friction swing phase

#### Warning !

In any circumstances never remove the screw (4).

### Setting stability (Screw 5)

- ⌚ Unscrew the screw (5) to increase the stability
- ⌚ Tighten the screw (5) to decrease the stability

#### Warning !

If the stability is decreased, the manual lock mechanism will not function.

- ⌚ Freeze the assembly by depositing Loctite® 242 (Ref. : Z31-242) on the screws
- ⌚ Use torque wrench (Ref. : Z0C102)

## Manual lock

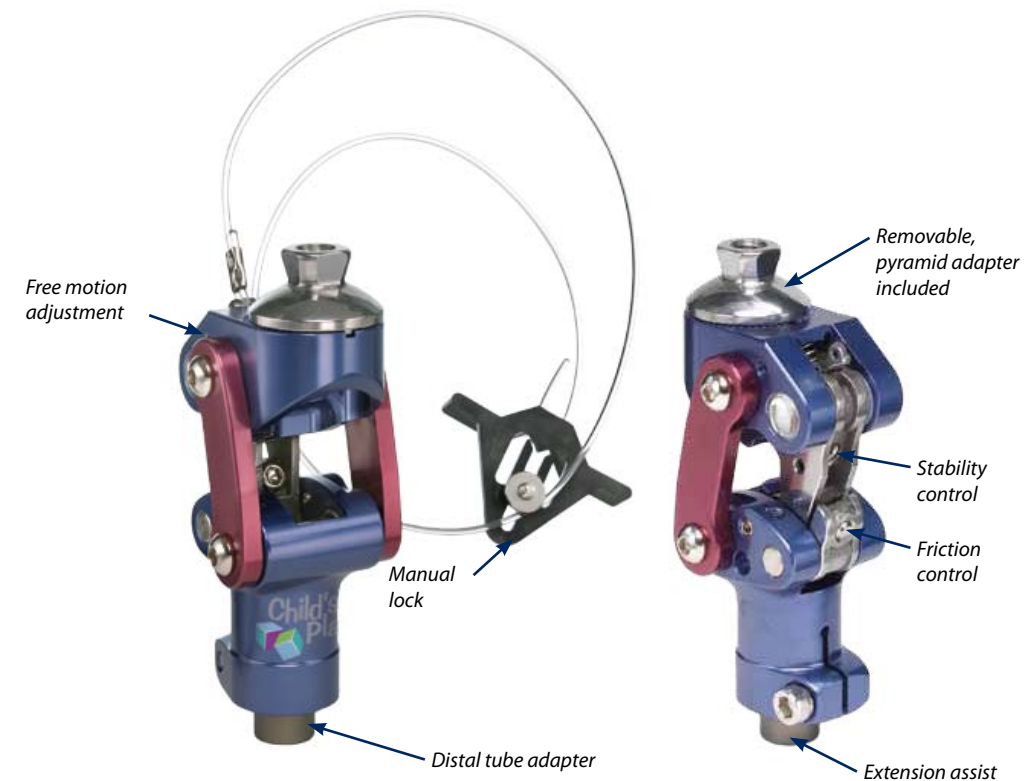
The knee C29 is equipped with a lock that can be manually released. The knee can be used for walking locked and unlocked during knee flexion (sitting). The knee can also be used for free walk.

### Adjusting bolt

- ⌚ After the fitting of the knee prosthesis and settings, locate an appropriate location on the socket for the locking mechanism handle and attaching it to the socket with the screw provided
- ⌚ Unscrew the 2 screws (7) on the mechanical handle
- ⌚ Adjust the cable tension by pulling the cable taut leaving minimal slack
- ⌚ Tighten the 2 screws (7) controlling the operation of the lock
- ⌚ Trim the excess cable if necessary

### Free knee joint setting

- ⌚ Unlock the knee
- ⌚ Tighten the screws (6) anti-lock
- ⌚ Use Loctite® 242 (Ref. : Z31-242) on the screw to prevent back

## Setting

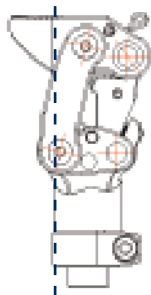
### Assembly

- ⌚ Cut the tube (Ref. : B2530) to the desired length
- ⌚ Chamfer the pipe
- ⌚ Place the tube in place and tighten the screw (1) to the specified torque
- ⌚ During the final assembly using thread lock on the screw (1)

### Alignment

Observe the alignment in the sagittal plane following :

- ⌚ The load line should pass through the center of the front axle.

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