## Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screws	Torques (Nm)
V02I-0620 (3)	10
V03TI-0616 (4)	5
V02I-0825 (5)	15

- ⌚ Assemble the screws using normal (Ref. : Z31-243) locking compound
- ⌚ Mount the tube on the stop of the bore bottom
- ⌚ The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ⌚ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ⌚ All assembly and adjustment work must be carried out by qualified personnel

## Cleaning / Storage

- ⌚ Stock the product in a dry place, in its original packing (on which the batch number is marked)
- ⌚ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- ⌚ Use a damp cloth to clean the product

## Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

## EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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### Flexion/extension control

Adjustment of swing phase :

- Regulator (2) : screw (clockwise) raises the extension's return resistance

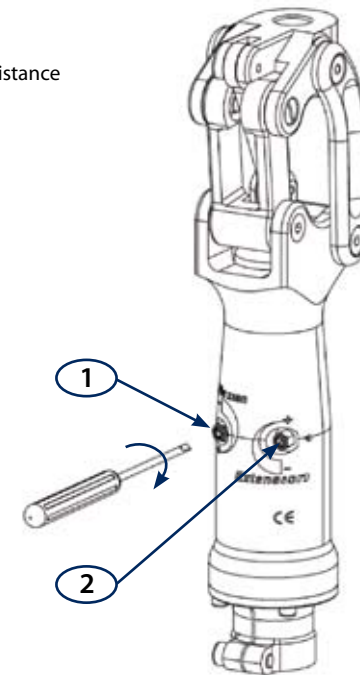
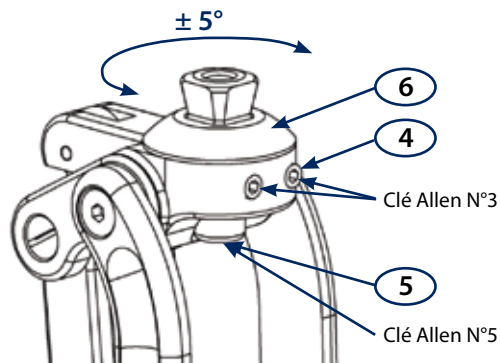
Adjustment of stance phase :

- Regulator (1) : screw (clockwise) raises the flexion's resistance


**When screwing regulator, never tighten the screw completely.**

### Adjustment of proximal attachment

Angulation maximum :  $\pm 5^\circ$

## Spares parts

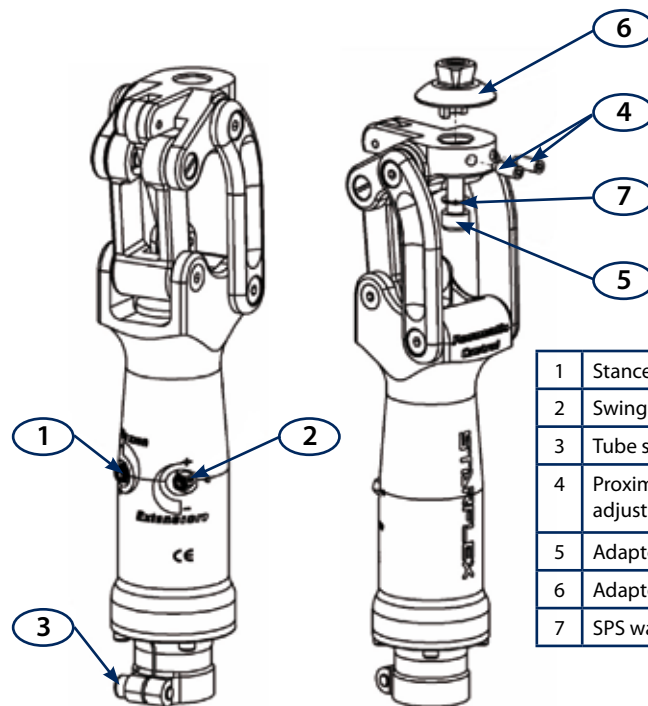1	Stance phase regulator	
2	Swing phase regulator	
3	Tube screw	V02I-0620
4	Proximal attachment adjustment screw	V03TI-0616
5	Adaptor screw	V02I-0825
6	Adaptor	
7	SPS washer	V21I-081313

## Compatibility

- Proximal attachment : connector of the STARFLEX range
- Distal attachment : Ø 30 mm tube

### Connectors in the STARFLEX range (upper section)

CA1  
Male pyramid


CA3  
Connector  
for temporary fitting

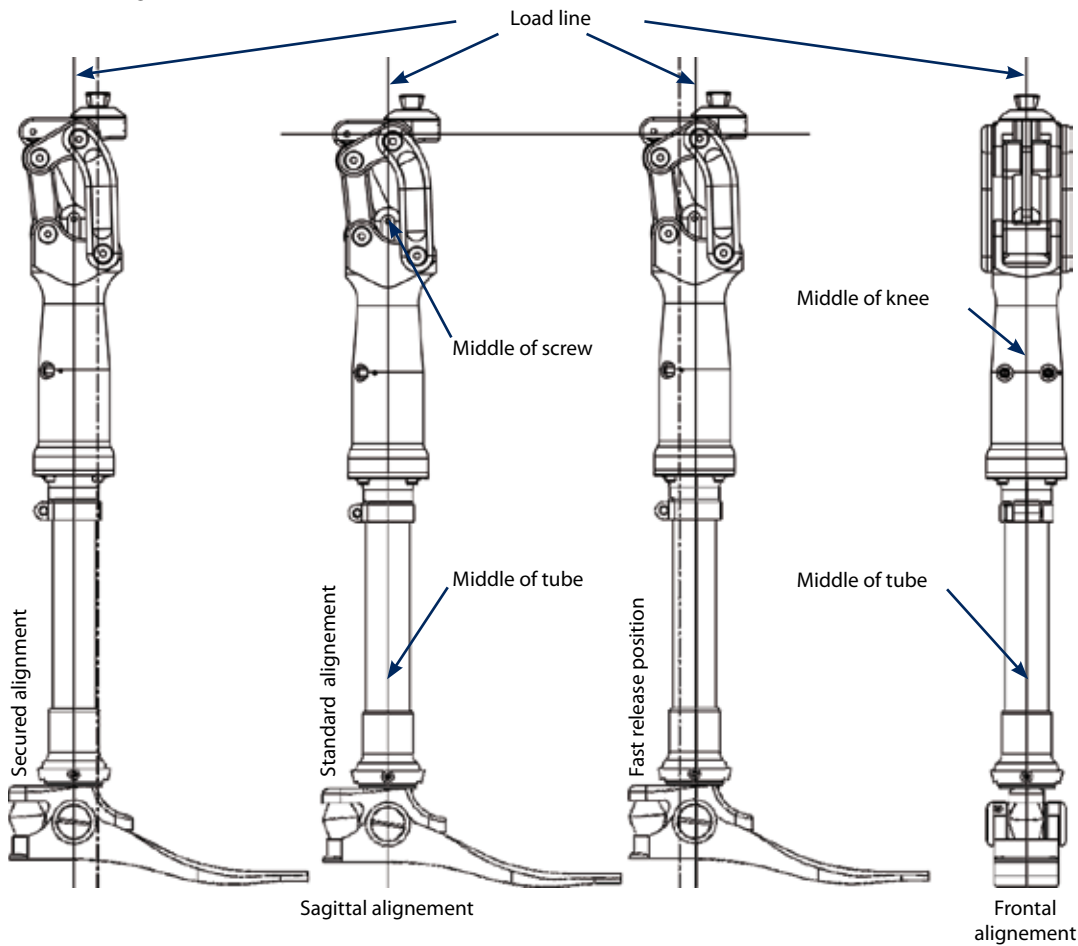CA7  
Rotary adaptor with anchor  
for cementing, large branches

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