
Fitting

Tilt amplitude : +/- 9° compared to the reference axis.

With C3220STFX/STMX : set the desired angulation using the 4 screws, in accordance with the limits set.

Spares parts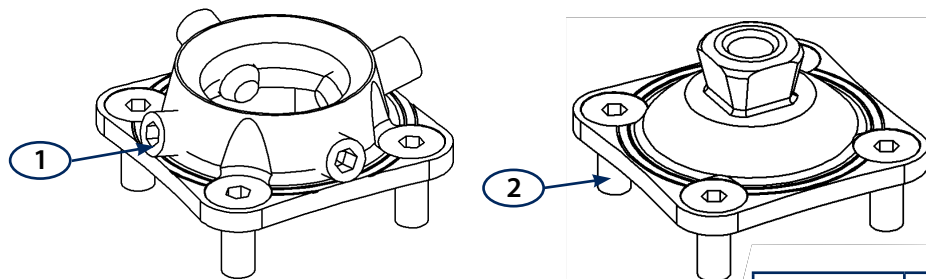

	C3220STFX	C3220STMX	
1	Flat point screw M8 L16	V03I-0816	V03Z-0816
		C39STFX	C39STMX
2	FHC screw M6 L16	V05I-0616	V05Z-0616

Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screws	Torques (Nm)
V03I-0816	13
V05I-0616	9

Screws	Torques (Nm)
V03Z-0816	15
V05Z-0616	9

- Assemble the screws using normal (Ref. : Z31-243) locking compound
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

JANTON hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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