
## Fitting

Tilt amplitude : +/- 9° compared to the reference axis.

With C3220STFX/STMX : set the desired angulation using the 4 screws, in accordance with the limits set.

## Spares parts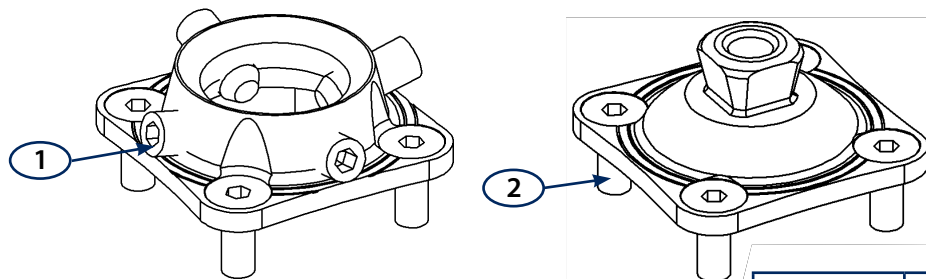

	C3220STFX	C3220STMX	
1	Flat point screw M8 L16	V03I-0816	V03Z-0816
		C39STFX	C39STMX
2	FHC screw M6 L16	V05I-0616	V05Z-0616

## Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screws	Torques (Nm)
V03I-0816	13
V05I-0616	9

Screws	Torques (Nm)
V03Z-0816	15
V05Z-0616	9

- ↪ Assemble the screws using normal (Ref. : Z31-243) locking compound
- ↪ The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ↪ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ↪ All assembly and adjustment work must be carried out by qualified personnel

## Cleaning / Storage

- ↪ Stock the product in a dry place, in its original packing (on which the batch number is marked)
- ↪ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

## Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

## EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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