
Set the desired tilt using the 4 screws, in accordance with the limits set.  
 Lock the assembly using LOCTITE 243 (ref Z31-243) locking compound on the screws V03I-0812 only.  
 Screw tightening torque (use key ZOC102) :

Screws	Torque (Nm)
V03I-0812	5
VO3PI-0516	3
V02Z-0516	9

## Spares parts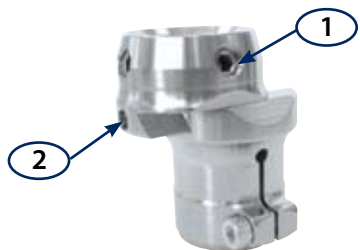

1	Flat point screw M8 L12	V03I-0812
2	Screw cone point M5 L16	V03PI-0516
3	Flat point screw M5 L16	V02Z-0516

## Restrictions on use / Precautions for use

- Assemble the screws using normal LOCTITE 243 locking compound
- The adaptor must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use (patient's activities, environment, etc...)
- All assembly and adjustment work must be carried out by qualified personnel

## Cleaning / Storage

- Store the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- Use a damp cloth to clean the product

## Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B.: In the event of a complaint, please enclose a return form correctly filled in; incomplete forms cannot be dealt with.*

## EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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TITANE

## Designation

Sliding double receiver in titanium.

- ↪ Weight : 149 g

## Purpose

Structural element used to make endoskeletal prostheses of the lower limbs for patients having average levels of activity and whose weight does not exceed 136 kg.

## Compatibility

The tubular adaptors are compatible with all the tubes and adaptors in the Titanium range.

## Fitting

### Adjustment

- ↪ Tilt amplitude: +/- 9° compared to the reference axis
- ↪ Offset amplitude: +/- 10 mm

### Offset adjustment

- ↪ The patient should not be put in charge of the prosthesis
- ↪ Use a 2,5 mm hex key to adjust the translation

### Attention !

Never remove the safety-lock set screw V03PI-0516.