Fitting

Laminating procedure on the anchor to be cemented

Two-phase lamination

- Put the positive element in a vacuum chamber
- Isolate the plaster using a PVA bag
- Prepare the lamination in the usual way
- Reinforce the distal section where the anchor to be cemented will be located
- Put the second PVA bag in place
- Carry out the first lamination cycle
- Once the resin has set, remove the PVA bag
- Place the unit on a transfer station
- Mark the alignments, and bend to shape the branches of the anchor to be cemented if necessary
- Cement the anchor using spatulating cement
- Once the cement has set, position the unit with the cemented anchor in the vacuum chamber
- Carefully isolate the pyramide of the anchor to be cemented, using lamination cap (Ref. : CA005)
- Reinforce the anchor branches
- Apply the finishing layer
- Carry out the lamination procedure
- Once the resin has set, free the threads, and remove the socket from the plaster
- Carry out the finishing work and fit the unit on the prosthesis

Adjustments

Use screws of the adapter (see technical instructions for the adapter).

Spare parts

Ref.	Description	Part Reference
1	Lamination cap	CA005

Restriction on use / Precautions for use

- The branches on the anchor to be cemented must be bent under cold conditions to shape regularly and progressively by hand, using a claw
- The cracking of the branches during bending is normal (related to the nature of alloy)
- The adaptor must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- Store the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- Use a damp cloth to clean the product

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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