Cleaning / Storage

- Store the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- Use a damp cloth to clean the product

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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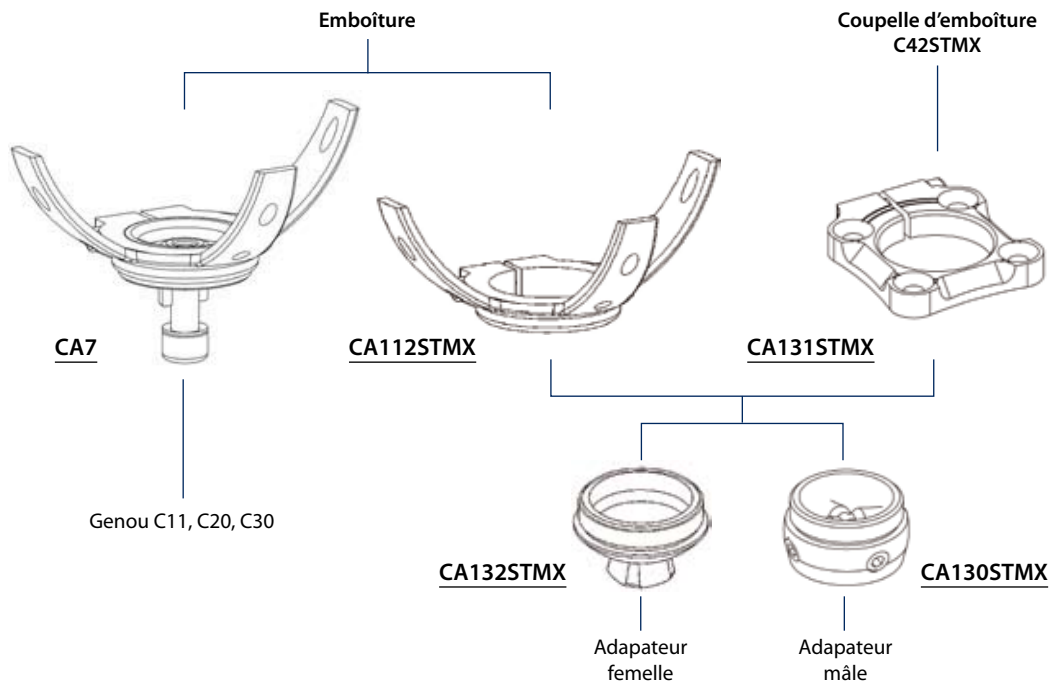
- Carry out the lamination procedure
- Once the resin has set, free the threads, and remove the socket from the plaster
- Carry out the finishing work and fit the unit on the prosthesis

Adjustments

Connector orientation

- Screw the connectors fully onto the CA112STMX anchor or CA131STMX four-hole plate
- Adjust the rotation by unscrewing the connector ¼ turn at most
- To maintain the desired orientation, tighten the locking screw (3) on the plate or anchor

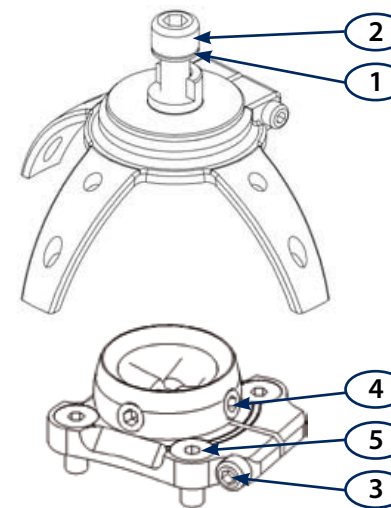
Pyramidal adjustment

Tilt amplitude : +/-9° compared to the reference axis.

Set the desired tilt using the 4 screws, in accordance with the limits set.

Lock the assembly using (Ref. : Z31-243) locking compound on the screws.

Spare parts

1	SPS washer	V21-081313
2	CHC screw M8 L25	V02I-0825
3	CHC screw M5 L30	V02Z-0530
4	BPL screw M8 L16	V03Z-0816
5	FHC screw M6 L16	V05Z-0616

Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screws	Torques (Nm)
V03Z-0816	15
V02Z-0530	6
V05Z-0616	9
V02I-0825	15

- The branches on the anchor to be cemented must be bent under cold conditions to shape regularly and progressively by hand, using a claw
- The cracking of the branches during bending is normal (related to the nature of alloy). Before forming the arms of the anchor screw the adapter so that it lays flat and tighten the screw at correct torque
- The adaptor must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

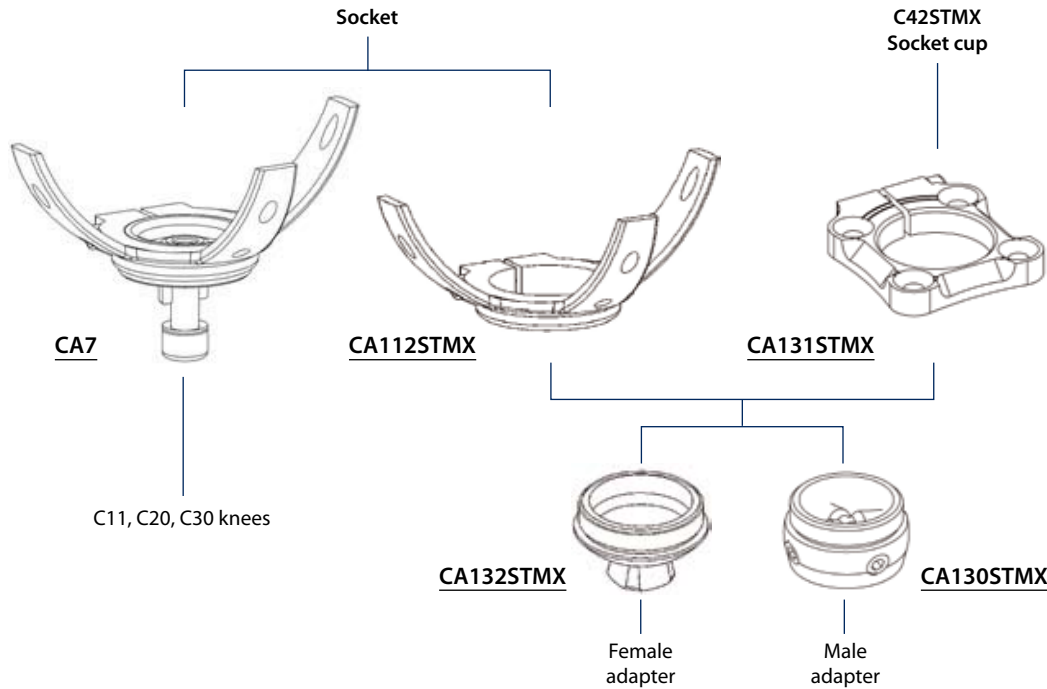
Purpose

Structural element used to make endoskeletal prostheses of the lower limbs

- Tested in accordance with ISO 10328 P6 136 kg

Compatibility

- All the adapters in the STARMAX range

Fitting

To adapt a CA7 connector to a knee in the STARFLEX range

See the technical instructions for the knee

Laminating procedure on the anchor to be cemented

Two-phase lamination

- Put the positive element in a vacuum chamber
- Isolate the plaster using a PVA bag
- Prepare the lamination in the usual way
- Reinforce the distal section where the anchor to be cemented will be located
- Put the second PVA bag in place
- Carry out the first lamination cycle
- Once the resin has set, remove the PVA bag
- Place the unit on a transfer station
- Mark the alignments, and bend to shape the branches of the anchor to be cemented if necessary. (Before forming the arms of the anchor screw the adapter so that it lays flat and tighten the screw at correct torque).
- Cement the anchor using spatulating cement
- Once the cement has set, position the unit with the cemented anchor in the vacuum chamber
- Carefully isolate the pyramide of the anchor to be cemented, using lamination cap (Ref. : CA004)
- Reinforce the anchor branches
- Apply the finishing layer

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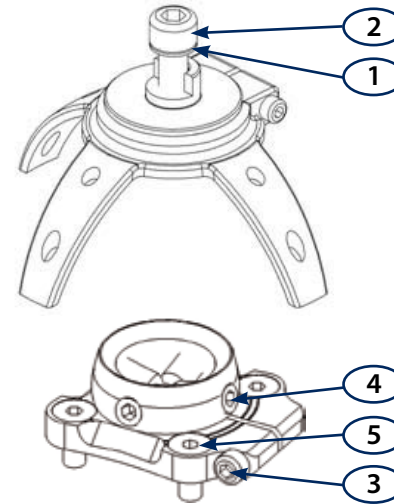
Designation

Set of rotational adapters

- CA7 : Anchor for cementing with large branches for knees of STARFLEX range
Weight : 190 g
- CA112STMX : Anchor for cementing with large branches, for rotational systems
Weight : 109 g
- CA113STMX : Anchor for cementing with large branches, with female rotational connector
Weight : 190 g
- CA114STMX : Anchor for cementing with large branches, with pyramid type rotational connector
Weight : 167 g
- CA130STMX : Female rotational connector
Weight : 80 g
- CA131STMX : Rotational plate with 4 holes
Weight : 104 g
- CA132STMX : Pyramid type rotational connector
Weight : 58 g