Adjustments

Rotation

- Loosen screw (6), adjust with screws (4) tilt maxi $\pm 5^\circ$
- Tighten screw (6)

Translation/tilt


- Loosen screw (7) and screw (1)
- Adjust
- Tighten screw (1) and screw (7)

Caution : always tighten screw (6) before screw (1)

Spares parts

1	Screw M8 L20
2	Plate washer $\varnothing 8$
3	Socket
4	Screw STHC M6 L16
5	Washer SPS $\varnothing 8$
6	Screw CHC M8 L20
7	Screw BPL M4 L5

The knee and screws (4) are not provided with the adapter.

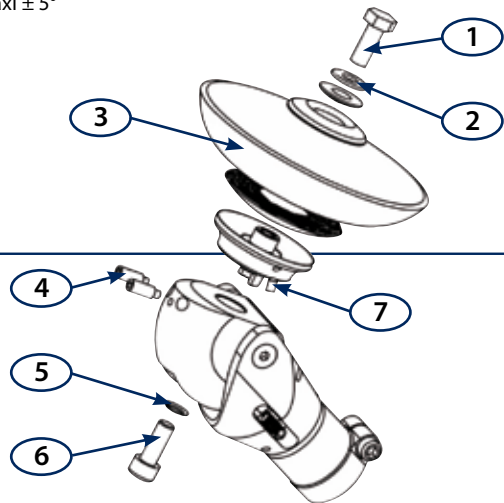Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screw	Torques (Nm)
V12-820 (1)	15
V03TI-0616 (4)	5
V02Z-0820 (6)	15
V03I-0405 (7)	4

- Assemble the screws using normal (Ref. : Z31-243) locking compound
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

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Designation

Adapter of STARFLEX range for temporary knee socket, made of light alloy

- ↪ Weight : 118 g

Purpose

This adapter is used for the production of endoskeletal lower limb prosthesis

- ↪ Weight limit : 100 kg

Compatibility

- ↪ Distal attachment : knee C11, C20 and C30

Fitting

- ↪ Rectify your positive according to your usual technique
- ↪ Extend the positive about 2 cm and realize a flat of a diameter $\varnothing 60$ mm minimum
- ↪ Thermoform the thermoplastic socket. Take out the socket of the positive
- ↪ Drill a hole of diameter 10 mm in the bottom of the socket
- ↪ Assemble the adapter in the socket
- ↪ Realise a windowing to be able to tighten the screw (1) with a latchkey