## Adjustments

### Rotation

- Loosen screw (6), adjust with screws (4) tilt maxi  $\pm 5^\circ$
- Tighten screw (6)

### Translation/tilt


- Loosen screw (7) and screw (1)
- Adjust
- Tighten screw (1) and screw (7)

**Caution :** always tighten screw (6) before screw (1)

## Spares parts

1	Screw M8 L20
2	Plate washer $\varnothing 8$
3	Socket
4	Screw STHC M6 L16
5	Washer SPS $\varnothing 8$
6	Screw CHC M8 L20
7	Screw BPL M4 L5

The knee and screws (4) are not provided with the adapter.

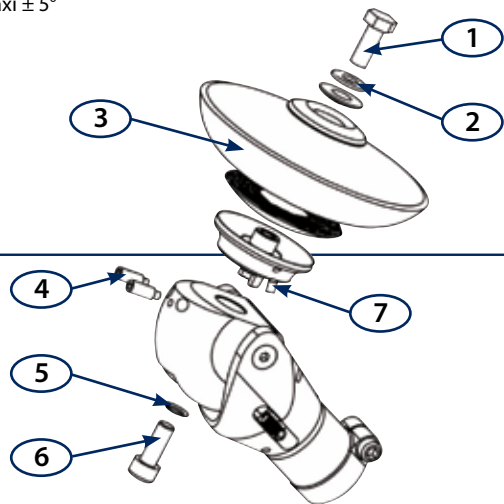## Restriction on use / Precautions for use

Tightening torques (use key (Ref. : ZOC102))

Screw	Torques (Nm)
V12-820 (1)	15
V03TI-0616 (4)	5
V02Z-0820 (6)	15
V03I-0405 (7)	4

- Assemble the screws using normal (Ref. : Z31-243) locking compound
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

## Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

## Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

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## Designation

Adapter of STARFLEX range for temporary knee socket, made of light alloy

- ↪ Weight : 118 g

## Purpose

This adapter is used for the production of endoskeletal lower limb prosthesis

- ↪ Weight limit : 100 kg

## Compatibility

- ↪ Distal attachment : knee C11, C20 and C30

## Fitting

- ↪ Rectify your positive according to you usual technique
- ↪ Extend the positive about 2 cm and realize a flat of a diameter Ø 60 mm minimum
- ↪ Thermoform the thermoplastic socket. Take out the socket of the positive
- ↪ Drill a hole of diameter 10 mm in the bottom of the socket
- ↪ Assemble the adapter in the socket
- ↪ Realise a windowing to e able to tighten the screw (1) with a latchkey