## Fitting

- Grease these contacts area (Réf. : Z10)
- Put nut lock glue (Réf. : Z31-243) on the ranging screw, cut it down the joint cap


To optimize your assembly work, we recommend using these parts in conjunction with other parts of the same class in the JANTON range.

## Adjustments

When making the orthosis, it is highly advisable to optimize joint alignment to reduce wear of the contact surfaces.

**NOTE :** Control joint parallelism in order to reduce wear of surfaces in contact.

## Spares parts

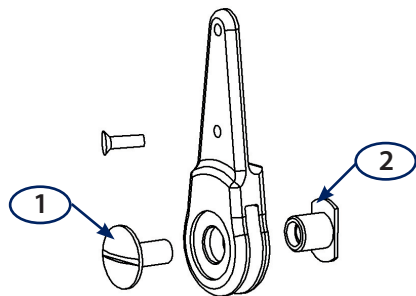		L14	L15	L16
1	Screw	V0013	V0016	V0016
2	Joint cap	V0012	V0015	V0015

## Restriction on use / Precautions for use

- Do not heat the joints
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

## Cleaning / Storage

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

## Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

## EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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