Composition

2 metal fittings (internal and external).

Spares parts

	M52	M53	M54
Articulation screw	V0108	V0107	V0107
Ball bearings	V406196	V405165	V405134

Restriction on use / Precautions for use

- ⌚ The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- ⌚ Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- ⌚ All assembly and adjustment work must be carried out by qualified personnel
- ⌚ Heat the part with a blowtorch before modifying its shape (any cold shaping will nullify the guarantee)

Cleaning / Storage

- ⌚ Stock the product in a dry place, in its original packing (on which the batch number is marked)
- ⌚ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

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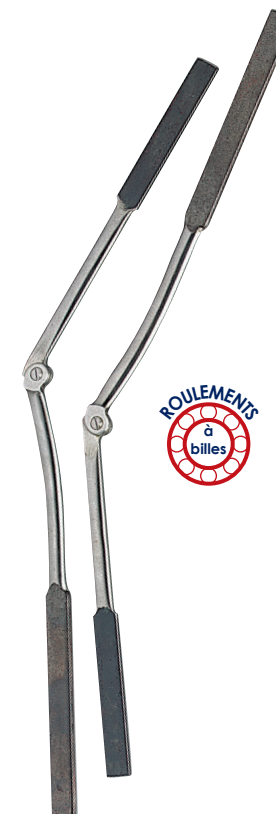
Designation

Fitting in semi-hardened steel for walking apparatus, covered head with no lock, upper and lower uprights with reserve.

- M52: Strong series for an upright 22 x 5 mm section
Reserves: 30 x 5,5 mm Ball bearings: 6 mm
- M53: Light series for an upright 20 x 4 mm section
Reserves: 28 x 4,5 mm Ball bearings: 5 mm
- M54: Child series for an upright 18 x 4,5 mm section
Ball bearings: 4 mm

Purpose

Knee articulation used for the manufacture of orthosis for a lower limb.