1	Articulation screw	V0159-1256
2	Bronze ring	M9108-1
3	FHC screw M4 L10	V05I-0410
4	FHC screw M4 L8	V05I-0408
5	FHC screw M3 L8	V05I-0308
6	Convex cruciform screw	V52I-0306

**Compatibility**

- All the orthotics components of JANTON range

**Fitting**

On the adjustable splint, the assembly between the leg part and the dial must be realized with the screw (6).

**Adjustment**

On the adjustable splint, the two M3 threaded holes made in the amount of leg allows adjustment every 6°. Amplitude : 6° recurvatum at 36° flexion (8 positions).

**Restriction on use / Precautions for use**

- Don't heat the knee joint
- Assemble the screws using normal Z31-243 (LOCTITE 243) locking compound
- Take care not to deform the mechanism during the bending
- The product must be used within the recommended weight limits and in accordance with the assessment made by an approved ortho-prosthetist depending on the patient concerned
- Product maintenance is subject to an assessment made by an approved ortho-prosthetist depending on the conditions of use
- All assembly and adjustment work must be carried out by qualified personnel

**Cleaning / Storage**

- Stock the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned


**Guarantee**

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

*N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.*

**EC markings**

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

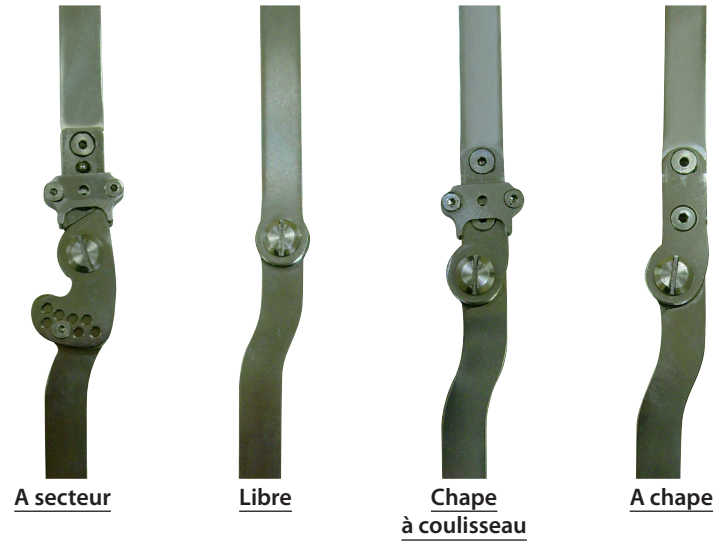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