Compatibility

Should be used for orthopaedic applications (standing/seating plastic shells...)

Spares parts

	Size 1	Sizes 2/3
Strap	X12xx-25	
Adjustment loops	X17-25	
Automatic buckles	X15-N25	X16-N25

Fitting

It is recommended to make a first fitting of the chestshoulderpad in order to precisely position the fastening point.

Restrictions one use / Precautions for use

- Machine-washable at 30° C 
- Do not :
 - Dry-clean 
 - Iron 
 - Use chlorinated agents 
 - Use in a clothes dryer 
- The chestshoulder pad must be used within the advised weight limits and in accordance with the judgement of the certified prosthetist concerning the patient to be fitted
- The maintenance of the product is decided by the prosthetist according to the type of use
- Assembly and adjustment must be performed by qualified personnel

Cleaning / Storage

- Store the product in a dry place, in its original packing (on which the batch number is marked)
- The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned

Guarantee

The guarantee is valid for six months as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B.: In the event of a complaint, please enclose a return form correctly filled in; incomplete forms cannot be dealt with.

EC markings

Groupe G2M, in its capacity as distributor, declares that all the parts conform to directive 2007/47/CE.

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