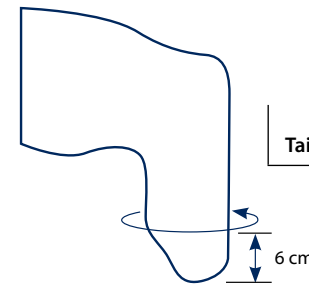
Désignation

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Cleaning process

Daily washing

- ↪ Hand wash the liner with warm water and neutral soap
- ↪ Never antibacterial soap
- ↪ Wash insid and outside of liner, rinse thouroughly with warm water, pat dry. Soapy residue remaining on liner may cause dermatitis - rinse inside of liner once or twice weekly to remove soap residue
- ↪ Place liner on towel with foam insert and allow to air dry (use foam onsert to retain liner shape when not in use)

Weekly washing

- ↪ Place a small amount of ribbing alcohol on a soft, clean cloth. Wipe the liner with a cloth thouroughly, wipe dry a towel. **Do not soak the liner in alcohol.**

To remove a stain on the fabric side, use a cloth dampened in nail polish remover (acetone) and rub until clean. **Do not soak in acetone or you will dissolve the gel.**

Guarantee

The guarantee is valid for one year as from the date of purchase of the product, if the product has been used in accordance with the manufacturer's instructions.

N.B. In the event of a complaint, please enclose a return form correctly filled in, incomplete forms cannot be dealt with.

EC markings

Groupe G2M hereby declares, in its quality as the manufacturer, that all its parts comply with directive 93/42/EC.

Compatibility

The XA39DT liners are appropriately for put in distal part a serrated from part of distal attachment system. A reducer is provided to adapt liners with those of the JANTON range in identical class.

Tightening

We recommend a tightening of 1,5 cm.

This datum is for the appreciation of the technician according to the form and the characteristic of the patient.

Setting up

Donning and doffing of silicon gel liner

- ⌚ Invert liner fully
- ⌚ **Avoid petroleum jelly and other oil based lubricants**, as they will damage the gel liner
- ⌚ Place liner with shuttle umbrella on a pre-supposed central area of the distal residuum
- ⌚ Talcum powder will form a scaling on the surface of the liner
- ⌚ Remove any air pockets trapped between the residual limb and the liner
- ⌚ **Do not pull the liner on**
- ⌚ Roll of for easy removal. **Do not pull the liner on**

Processing

In order to optimise your assemblies, we recommend that use the liners in conjunction with those of the JANTON range in identical class.

Restriction on use / Precautions for use

- ⌚ Do not pull liner off
- ⌚ These parts are to be used within the limits of recommended weight and at the ortho-prosthetist's discretion, according to the activity of the patient to be fitted
- ⌚ The maintenance of the part remains at the discretion of the ortho-prothesist, in accordance with the framework of use (activity of the patient, the environnement, etc...)
- ⌚ Qualified staff must carry out the assembly and adjustment
- ⌚ Liner should never be installed on a stub with not healed wounds

Cleaning / Storage

- ⌚ Store the product in a dry place, in its original packing (on which the batch number is marked)
- ⌚ The frequency of inspections to check the equipment is subject to the assessment made by the ortho-prosthetist depending on the patient concerned
- ⌚ Always keep the foam insert the liner when not in use. Lying flat for long periods may cause creases to develop

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
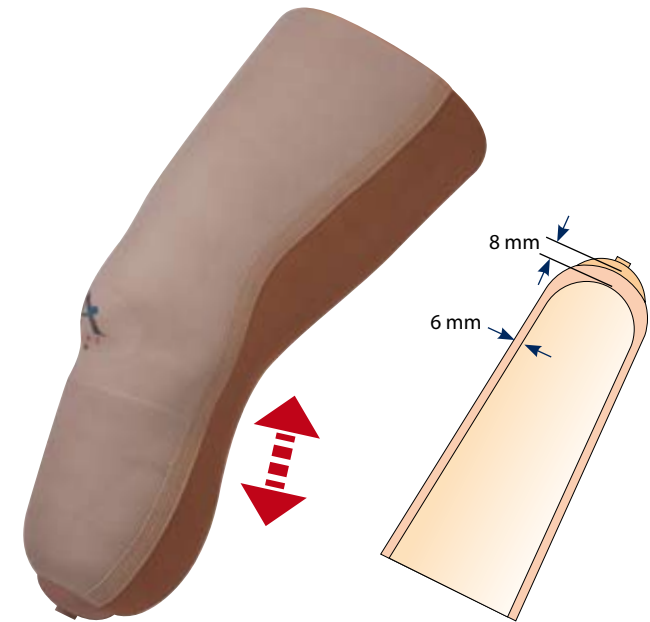
Designation

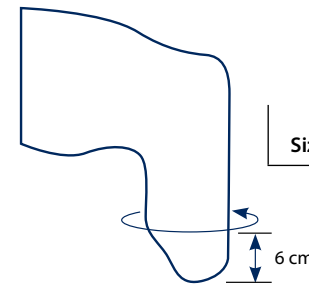Non sticky copolymer gel liner thick with distal attachment.
 Sensitive stump.
 To choose liner's size measure the limb size taken 6 cm above distal end :

Sizes	16 - 19 cm	20 - 23 cm	24 - 27 cm	26 - 32 cm	28 - 31 cm	32 - 35 cm
	16	20	24	26	28	32

Purpose

This liner is used for the production of lower limb prostheses.